



CONVENIENT, FAST, AND RELIABLE. GOOD THINGS DO COME IN LITTLE PACKAGES.

TEGO™ Bovine

Simple and Safe Blood Collection

SELF-CONTAINED SHARPS greatly reduces risk of needle stick injury

SHIP UNREFRIGERATED through regular postal mail

COMPATIBLE with existing lab processes

TEST FOR GENETICS, such as defects, trait selection, parentage and animal ID

TEST FOR PREGNANCY

TEST FOR DISEASES such as BVD and pestivirus

EFFICIENTLY STORE SAMPLES for years at room temperature without affecting DNA sample integrity

www.itlbiomedical.com



TEGO™ Bovine


Simple and Safe Blood Collection



TEGO™ was developed in close consultation with U.S. laboratories, veterinarians and livestock producers worldwide, and is designed to simplify and lower the risks associated with blood sampling, such as potential injury from needle sticks or agitated cattle.

PRODUCT USE

1

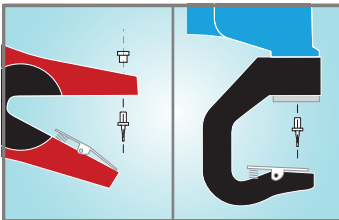

Begin by inserting TEGO™ Adapter Pin into ear-tagging applicators* – a step that is performed only once.

2



Remove the kit from packaging and place it into the ear-tagging applicator.

3



Position over the animal's ear (front or back), targeting the vein-rich area, then clamp.

4



Release the ear-tagging applicator, leaving the kit attached to the ear for passive blood collection (about 10-20 seconds).

5



Remove the kit from the ear by holding kit and pulling away.

6



Slide the blood collection card from the holder, and place it almost entirely into the TEGO™ envelope.

7



Label the exposed end of the card. After inserting the entire card into the TEGO™ envelope, place folded-over TEGO™ envelopes into a mailing envelope and mail to testing facility.

Note: Kit is compatible with Allflex® Total Tagger and Allflex® Retract-O-Matic™ ear-tagging applicators; **REQUIRES USE OF TEGO™ ADAPTER PIN.**



Product Codes

A100911 TEGO™ Bovine Blood Collection Kit
Available in units of 10, 50, 300
A100913 TEGO™ Adapter Pin 5/pack

Product Information

- Latex Free Kit
- Single Use Only



HUMAN INSPIRED. SCIENTIFICALLY ENGINEERED.

ITL BioMedical

www.itlbiomedical.com
Sales Office: 1.888.411.2851
sales.animalhealthcare@itlbiomedical.com