



CONVENIENT, FAST, AND RELIABLE. GOOD THINGS DO COME IN LITTLE PACKAGES.

TEGO™ Card

Simple and Safe Blood Collection

EASY TO APPLY blood samples to the card and immediately ship to the lab

SHIP UNREFRIGERATED through regular postal mail

COMPATIBLE with existing lab processes

TEST FOR GENETICS, such as defects, trait selection, parentage and animal ID

TEST FOR PREGNANCY

TEST FOR DISEASES such as BVD and PRRS

EFFICIENTLY STORE SAMPLES for years at room temperature without affecting DNA sample integrity

www.itlbiomedical.com



TEGO™ Card


Simple and Safe Blood Collection



TEGO™ was developed in close consultation with U.S. laboratories, veterinarians and livestock producers worldwide, and is designed to simplify and lower the risks associated with blood sampling.


PRODUCT USE

1

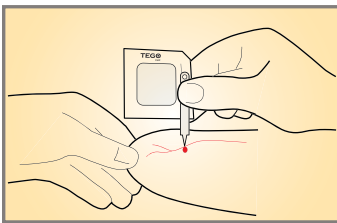


Use lancet to prick a vein on the animal's ear or other appropriate location; safely dispose the lancet or sharps device.

2

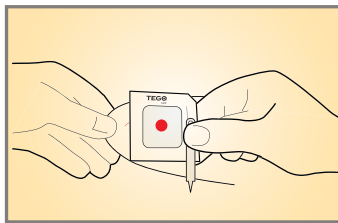


Allow the blood to form a drop, hold the card by the frame, and blot onto the center area.

3

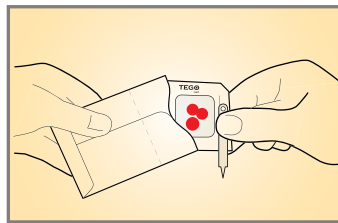

Place card into the provided sample envelope.

4

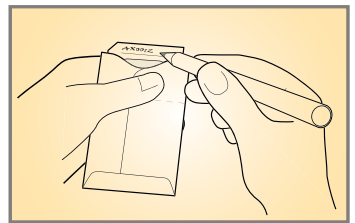

Label and insert into mailing envelope and drop into postal mail.

Product Codes

- A100915 TEGO™ Card 1/pack 1,000/Carton
- A100917 TEGO™ Card with Lancet 1/pack 1,000/Carton
- A100919 TEGO™ Card 10/pack 1,000/Carton
- A100920 TEGO™ Card with Lancet 10/pack 1,000/Carton

Product Information

- Latex Free Kit
- Single Use Only



HUMAN INSPIRED. SCIENTIFICALLY ENGINEERED.

ITL BioMedical
www.itlbiomedical.com
Sales Office: 1.888.411.2851
sales.animalhealthcare@itlbiomedical.com