



CrocClamp™ Non-reopening Tubing Clamp

Clamp tubing easily and securely with non-reopening medical tubing clamps

This non-reopening tubing clamp is designed for use with blood and apheresis collection systems and other medical procedure kits where a permanent seal of a flexible tube is required.

- Ergonomic, easy to close
- Clamp closure aligns with tubing to ensure occlusion
- Non-reopening locking mechanism prevents accidental clamp release
- Color options distinguish from other clamps on the blood collection system. Now available in blue and red.



The standard for best practice for non-reopening tubing clamps.

Prevent accidental clamp release with non-reopening medical tubing clamps


CrocClamp™ Non-reopening Tubing Clamp:

- Ergonomic, easy to close
- Clamp closure aligns with tubing to ensure occlusion
- Non-reopening locking mechanism prevents accidental clamp release
- Compatible with whole blood and apheresis kit tubing
- Available in blue and red
- Single use clamp



Easy to use, permanent tubing clamp

1. Place fingers along the top and bottom of the CrocClamp™ so that they extend beyond the edge of the clamp.

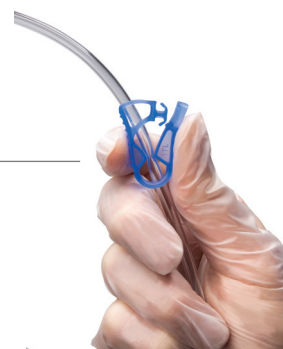

2. Squeeze together the top and bottom of the CrocClamp™ to ensure full closure of the clamp.




3. CrocClamp™ in locked position.



Multiple colours available

Distinguish it from other clamps in your blood collection system.

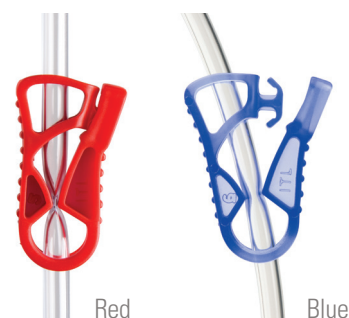

Visit itlbiomedical.com to request your samples today

Product Code:

A100111 (Blue): Bulk, Non-sterile, 4,000/case

A100111R (Red): Bulk, Non-sterile, 4,000/case

(both suitable for ETO and steam sterilization)

Product Information:



Not Made with Natural Rubber Latex



Single Use Only

