



TRAKE-fit Collar Set

Tracheostomy tube holder for maximum patient comfort

A recent study¹ compared the rates of skin-related complications and accidental decannulation in patients who received hook and loop securement and twill ties.

The use of hook and loop securement was associated with a decrease in the rate of skin irritation (OR: 0.41;95% CI: 0.17- 0.97; P=0.039) and there were no accidental decannulation events.

TRAKE-fit Tracheostomy Tube Holder for when securement matters.

Partners in Healthcare Associated Blood Stream
Infection Prevention Products and Strategies



Keeping the tracheostomy tube clean, dry and in place

- Tracheostomy is one of the most common procedures for long-term mechanical ventilation secondary to chronic respiratory failure¹.
- Advantages of a tracheostomy include facilitating oral hygiene, promoting patient comfort, providing a more secure airway and decreasing the risk of tracheal necrosis.
- Peristomal care should be performed at least every 8 hours with change to dressing as needed.
- The collar should be changed if wet or soiled. Assess the skin under the collar for any abnormalities.



For Maximum patient comfort

The TRAKE-fit Tracheostomy Tube Holder are available in single or double collar sets.

- Quick and easy to fit in seconds
- Remains secure once fitted
- Maximum comfort with cotton-lined neckband
- No adhesive, no narrow tape
- Completely adjustable
- Narrow tabs available for pediatric and small adult use



- Patient should be constantly monitored while this product is in use.
- TRAKE-fit is single use only and supplied non sterile.
- Patient should be constantly monitored while this product is in use.
- For 25/43-1850 and 25/43-1862, when the neckband becomes soiled, replace it with the extra one from the pack.

References

1. Data on File

For further information or samples, visit itlbiomedical.com or contact your local sales office.

Product Codes

25/43-1850	Double Tracheostomy Collar Set, non sterile (Each Set includes (2) neckbands, (2) wide hook/loop tabs)	25/carton
25/43-1851	Single Tracheostomy Collar Set, non sterile (Each Set includes (1) neckband, (2) wide hook/loop tabs)	25/carton
25/43-1862	Double Tracheostomy Collar/Narrow Tab Set, non sterile (Each Set includes (2) neckbands, (2) narrow hook/loop tabs)	25/carton

Product Information



www.itlbiomedical.com/symbolglossary